HELSE MØRE OG ROMSDAL

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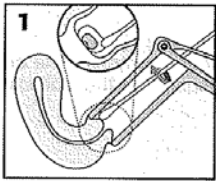
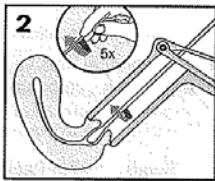
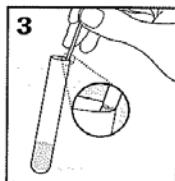
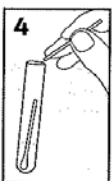
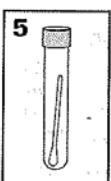
For **cervix** pensel gjeld denne rettleiing:

ENDOCERVICAL SWAB SPECIMEN COLLECTION

WARNING: DO NOT PRE-WET SWAB IN cobas® PCR MEDIA BEFORE COLLECTION!

 <p>1. CLEAN: Using one of the swabs (provided), remove excess mucus from the cervical os and surrounding mucosa. Discard the swab after use. NOTE: Cleaning excess mucus from the cervical os is required to assure an adequate sample is obtained for processing. A large-tipped cleaning swab, such as Puritan 25-808 1PR (not provided) can be used.</p>	 <p>2. COLLECT: To collect the specimen, insert the other provided swab into the endocervical canal. Gently rotate the swab 5 times in one direction in the endocervical canal. Do not over-rotate. Carefully withdraw the swab, avoiding any contact with the vaginal mucosa.</p>	 <p>3. ALIGN: Remove the cap from the cobas® PCR Media tube and lower the swab specimen into the tube until the visible dark line on the swab shaft is aligned with the tube rim. The tip of the swab should be just above the media surface near the hexagonal Roche logo.</p>	 <p>4. BREAK: Carefully leverage the swab against the tube rim to break the swab shaft at the dark line; discard the top portion of the swab.</p>	 <p>5. CLOSE: Tightly re-cap the cobas® PCR Media tube. The specimen is now ready for transport.</p>
---	--	---	--	--

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